Science **made** smarter

Instructions for Use

# VRA201

EN	Instructions for Use
DE	Gebrauchsanweisung
SV	Bruksanvisning
DA	Brugsanvisning
NO	Bruksanvisning
FR	Instructions d'utilisation
ES	Manual de instrucciones
IT	Istruzioni per l'uso
PL	Instrukcja obsługi
CS	Návod k použití
SL	Navodila za uporabo sistema
EL	Οδηγίες χρήσης
TR	Kullanım Talimatları
BRPT	Instruções de Uso
KO	사용설명서 - 한국어
JA	取扱説明書
RU	Инструкция по эксплуатации
ZH	使用说明

85214.12 D-0125524-B – 2022/08

**Interacoustics**

# Instructions for Use - EN

# Table of contents

<b>1</b>	<b>Introduction .....</b>	<b>1</b>
1.1.	About this manual.....	1
1.2.	Intended use.....	1
1.3.	Contraindications.....	1
1.4.	Product description.....	1
1.5.	Warnings .....	2
<b>2</b>	<b>Unpacking and installation .....</b>	<b>3</b>
2.1	Unpacking and inspection .....	3
2.2	Reporting imperfections .....	3
2.3	Markings.....	4
2.4	Safety precautions.....	5
2.4.1	Cautions – general.....	5
2.4.2	Environmental factors .....	5
2.4.3	Electrical and electrostatic safety.....	6
2.4.4	Electromagnetic compatibility (EMC).....	6
2.4.5	Miscellaneous .....	6
2.4.6	Use of equipment after transport and storage .....	7
<b>3</b>	<b>Operating instructions .....</b>	<b>8</b>
3.1	How to operate VRA201 .....	8
3.2	Connection panel dictionary.....	8
3.3	Safety precautions when connecting the VRA201 .....	8
<b>4</b>	<b>Maintenance .....</b>	<b>9</b>
4.1	General maintenance procedures.....	9
4.2	Repair .....	9
4.3	Warranty.....	9
<b>5</b>	<b>General technical specification .....</b>	<b>11</b>
5.1	VRA – technical specifications .....	11
5.2	General information about specifications .....	11
5.3	Electromagnetic Compatibility (EMC) .....	12

# 1 Introduction

## 1.1. About this manual

This manual is valid for the Interacoustics VRA201. This product is manufactured by:

**Interacoustics A/S**  
Audiometer Allé 1  
5500 Middelfart  
Denmark  
Tel.: +45 6371 3555  
Fax: +45 6371 3522  
E-mail: [info@interacoustics.com](mailto:info@interacoustics.com)  
Web: [www.interacoustics.com](http://www.interacoustics.com)

## 1.2. Intended use

The Interacoustics VRA201 is a manually controlled visual reinforcement tool. The tool is aimed at giving visual rewards to pediatric patients in order to keep their focus during tests.

## 1.3. Contraindications

None known.

## 1.4. Product description

The Interacoustics VRA201 is a box with a toy animal hidden behind a dark plexiglass. By means of a push on a response switch, the toy animal is illuminated and starts moving.

This device thereby provides visual reinforcement for children that are tested for their hearing capability with illumination and movement.

### Included parts:

VRA201 box	
Power supply	UES24LCP-120200SPA
Patient response	APS3
Instructions for Use	Multilingual

## 1.5. Warnings

Throughout this manual, the following meanings of warning, caution and notice apply:

	The <b>WARNING</b> label identifies conditions or practices that may present danger to the patient and/or user.
	The <b>CAUTION</b> label identifies conditions or practices that could result in damage to the equipment.
NOTICE	<b>NOTICE</b> is used to address practices not related to personal injury.

## 2 Unpacking and installation

### 2.1 Unpacking and inspection

#### **Check box and contents for damage**

When you receive VRA201, please check the shipping box for rough handling and damage. If the box is damaged, you should keep it until the contents of the shipment have been checked mechanically and electrically. If your VRA201 is faulty, please contact your local distributor. Keep the shipping material for the carrier's inspection and insurance claim.

#### **Keep carton for future shipment**

VRA201 comes in its own shipping carton, which is specially designed for the VRA201 system. Please keep this carton. You will need it should you need to return VRA201 for service. If service is required, please contact your local distributor.

### 2.2 Reporting imperfections

#### **Inspect before connection**

Prior to connecting the product, you should inspect it once more for damage. You should visually check the cabinet and the accessories for scratches and missing parts.

#### **Report any faults immediately**

Any missing parts or malfunctions should be reported immediately to the supplier of the instrument together with the invoice, serial number, and a detailed report of the problem. At the end of this manual, you will find a "Return Report", where you can describe the problem.

#### **Please use the "Return Report"**

Use of the return report provides the service engineer with the relevant information to investigate the reported issue. Without this information, there may be difficulty in determining the fault and repairing the device. Please always return the device with a completed Return Report in order to guarantee a satisfactory repair.

## 2.3 Markings

The following markings can be found on the instrument:

Symbol	Explanation
	Type B applied parts. According to IEC 60601-1.
	Follow Instructions for Use.
	WEEE (EU-directive) This symbol indicates that when the end-user wishes to discard this product, it must be sent to separate collection facilities for recovery and recycling. Failing to do so may endanger the environment.
	The CE-mark indicates that Interacoustics A/S meets the requirements of Annex II of the Medical Device Directive 93/42/EEC. TÜV Product Service.
	Serial number.
	Date of manufacture.
	Manufacturer.
	Reference number.
	Keep dry.
	Transport and storage humidity range.
	Transport and storage temperature range.
	Logo.

## 2.4 Safety precautions

Read these instructions carefully and completely before using the product.

### 2.4.1 Cautions – general

If the system is not functioning properly, do not use it until all necessary repairs are made and the unit is tested for proper functioning in accordance with Interacoustics' specifications.

Do not drop or in any other way cause undue impact to this device. If the instrument is damaged, return it to the manufacturer for repair. Do not use the instrument if any damage is suspected.

This product and its components will perform reliably only when operated and maintained in accordance with the instructions contained in this manual, accompanying labels, and/or inserts. A defective product should not be used. Make sure all connections to external accessories are secured properly. Parts which may be broken or missing or are visibly worn or distorted should be replaced immediately with clean, genuine replacement parts manufactured by or available from Interacoustics. Interacoustics will make available on request circuit diagrams, component part lists, descriptions, calibration instructions, or other information that will assist authorized service personnel to repair those parts of this instrument that are designated by Interacoustics as repairable by service personnel.

No parts of the equipment can be serviced or maintained while in use with the patient.

Only connect accessories purchased from Interacoustics to the instrument. Only accessories, which have been stated by Interacoustics as compatible, may be connected to the device.

The instrument is not intended for use in environments exposed to fluid spills.

### 2.4.2 Environmental factors

Storage in temperatures outside the temperature range specified in sections 2.4.5 and 5.1 may cause permanent damage to the instrument and its accessories.

Do not use the device in the presence of fluid that can get in contact with any of the electronic components or wiring. Should the user suspect fluids have been in contact with system components or accessories, the unit should not be used until deemed safe by an authorized service technician.

Do not place the instrument next to a heat source of any kind, and allow enough space around the instrument to ensure proper ventilation.

### 2.4.3 Electrical and electrostatic safety

Do not modify this equipment without authorization from Interacoustics. Do not disassemble or modify the product as this may impact the safety and/or performance of the device. Limit servicing to qualified personnel.

For maximum electrical safety, turn off the power when it is left unused.

The power plug must be placed so it is easy to pull out the plug.

Do not use any additional multiple socket-outlet or extension cord. For safe setup, please refer to section 3.1.2.

Do not use the equipment if it is showing visible signs of damage.

The instrument is not protected against ingress of water or other liquids. If any spillage occurs, check the instrument carefully before use or return for service.

No part of the equipment can be serviced or maintained while in use with the patient.

### 2.4.4 Electromagnetic compatibility (EMC)

#### NOTICE

Although the instrument fulfills the relevant EMC requirements, precautions should be taken to avoid unnecessary exposure to electromagnetic fields, e.g., from mobile phones, etc. If the device is used adjacent to other equipment, it must be observed that no mutual disturbance appears. Please also refer to the appendix regarding EMC.

Use of accessories and cables other than specified, apart from cables sold by Interacoustics or representatives, may result in increased emission or decreased immunity of the equipment. For a list of accessories that fulfil the requirements, please refer to the appendix regarding EMC.

### 2.4.5 Miscellaneous

#### Please note:

Storage in temperatures below 0°C /32°F and above 50°C /122°F may cause permanent damage to the instrument and its accessories.

Do not place the instrument next to a heat source of any kind. Within the European Union, it is illegal to dispose of electrical and electronic waste as unsorted municipal waste. Electrical and electronic waste may contain hazardous substances and therefore must be disposed of separately. Such products will be marked with the crossed-out wheelie-bin image shown to the left. User cooperation is important in order to ensure a high level of reuse and recycling of electrical and electronic waste. Failure to recycle such waste products in an appropriate way may endanger the environment and consequently the health of human beings.

Outside the European Union, local regulations should be followed when disposing of the product after its end of life.

#### **2.4.6 Use of equipment after transport and storage**

Please make sure that the instrument is functioning correctly before use. If the instrument has been stored in a cold environment (even for a short period of time), please allow the instrument to become acclimatized. This can take long time depending on the conditions (such as environmental humidity). You can reduce the condensation by storing the instrument in its original packaging. If the instrument is stored under warmer conditions than the actual use conditions, no special precaution is required before use.

### 3 Operating instructions

#### 3.1 How to operate VRA201

Connect the medically approved power supply and the patient response to the VRA201. The toy and light are controlled manually by the means of the activator button. The speed of the toy can be adjusted by the means of a potentiometer on the rear panel.

#### 3.2 Connection panel dictionary

Name	Description
1	 Power - Only use power supply UES24LCP-120200SPA
2	Activator - Patient response used to activate the visual reinforcement
3	Intensity - Speed of the toy animal

#### 3.3 Safety precautions when connecting the VRA201


WARNING

WARNING

Only use power supply UES24LCP-120200SPA.  
The separable mains plug for UES24LCP-120200SPA is used to safely disconnect mains from the device. Do not position the power supply in a position so that it is difficult to disconnect the device.

Fig 1. VRA201 used with the medically approved power supply.

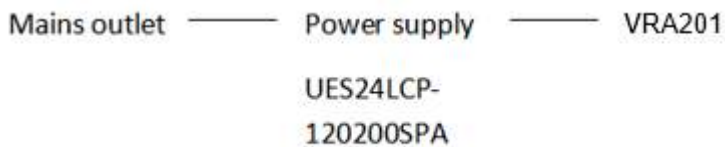## 4 Maintenance

### 4.1 General maintenance procedures

- Before cleaning, always switch off and disconnect from the power supply
- Use a soft cloth lightly dampened with a cleaning solution to clean all exposed surfaces
- Do not autoclave, sterilize or immerse the instrument or accessory in any fluid
- Do not use hard or pointed objects to clean any part of the instrument or accessory
- Do not let parts that have been in contact with fluids dry before cleaning

#### Recommended cleaning and disinfection solutions:

- Warm water with mild, non-abrasive cleaning solution (soap)
- 70% isopropyl alcohol only on hard cover surfaces

### 4.2 Repair

Interacoustics is only considered to be responsible for the validity of the CE marking, effects on safety, reliability and performance of the equipment if:

1. assembly operations, extensions, readjustments, modifications or repairs are carried out by authorized persons
2. the electrical installation of the relevant room complies with the appropriate requirements, and
3. the equipment is used by authorized personnel in accordance with the documentation supplied by Interacoustics.

It is important that the customer (distributor) fills out the RETURN REPORT every time a problem arises and sends it to

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This should also be done every time an instrument is returned to Interacoustics. (This of course also applies in the unlikely worst-case scenario of death or serious injury to a patient or user).

### 4.3 Warranty

Interacoustics guarantees that:

- The Interacoustics VRA201 is free from defects in material and workmanship under normal use and service for a period of 24 months from the date of delivery by Interacoustics to the first purchaser
- Accessories are free from defects in material and workmanship under normal use and service for a period of ninety (90) days from the date of delivery by Interacoustics to the first purchaser

If any product requires service during the applicable warranty period, the purchaser should communicate directly with the local Interacoustics service center to determine the appropriate repair facility. Repair or replacement will be carried out at Interacoustics' expense, subject to the terms of this warranty. The product requiring service should be returned promptly, properly packed, and postage prepaid. Loss or damage in return shipment to Interacoustics shall be at the purchaser's risk.

This shall apply solely to the original purchaser. This warranty shall not apply to any subsequent owner or holder of the product. Furthermore, this warranty shall not apply to, and Interacoustics shall not be responsible for, any loss arising in connection with the purchase or use of any Interacoustics product that has been:

- repaired by anyone other than an authorized Interacoustics service representative;
- altered in any way so as, in Interacoustics opinion, to affect its stability or reliability;
- subject to misuse or negligence or accident, or that has had the serial or lot number altered; defaced or removed; or
- improperly maintained or used in any manner other than in accordance with the instructions provided by Interacoustics.

In no event shall Interacoustics be liable for any incidental, indirect or consequential damages in connection with the purchase or use of any Interacoustics product.

This warranty is in lieu of all other warranties, express or implied, and of all other obligations or liabilities of Interacoustics. Interacoustics does not give or grant, directly or indirectly, the authority to any representative or other person to assume on behalf of Interacoustics any other liability in connection with the sale of Interacoustics products.

**INTERACOUSTICS DISCLAIMS ALL OTHER WARRANTIES, EXPRESSED OR IMPLIED, INCLUDING ANY WARRANTY OF MERCHANTABILITY OR FOR FUNCTION OF FITNESS FOR A PARTICULAR PURPOSE OR APPLICATION.**

## 5 General technical specification

### 5.1 VRA – technical specifications

General		
Standards:	Safety (power supply):	IEC 60601-1:2012
	EMC:	IEC 60601-1-2:2014
Operation environment:	Temperature:	15-35 °C
	Relative humidity:	30-90%
Transport & storage:	Storage temperature:	0°C-50°C
	Transport temperature:	-20-50 °C
	Rel. humidity:	10-95%
Power supply 	UES24LCP-120200SPA	Only use specified power supply unit type Input: 100-240VAC 50/60 Hz, 500mA Output: 12.0 VDC 2.0A
Dimensions	H x W x L	D28 x W30 x H40 cm (11x12x16 inches)
VRA201 weight		6.6 kg/14.7 lbs.
Light		Two LED strips of 8.5W total, 12 Vdc
Construction		Wooden (MDF) cabinet

### 5.2 General information about specifications

Interacoustics continuously strives to improve its products and their performance. Therefore, the specifications can be subject to change without notice.

Enquiries about representatives and products may be sent to:

**Interacoustics A/S**

Audiometer Allé 1  
5500 Middelfart  
Denmark

Tel.: +45 6371 3555

Fax: +45 6371 3522

E-mail: [info@interacoustics.com](mailto:info@interacoustics.com)

Web: [www.interacoustics.com](http://www.interacoustics.com)

### 5.3 Electromagnetic Compatibility (EMC)

- This VRA201 is suitable in hospital environments except for near active HF surgical equipment and RF shielded rooms of systems for magnetic resonance imaging, where the intensity of electromagnetic disturbance is high
- Use of this VRA201 adjacent to or stacked with other equipment should be avoided because it could result in improper operation. If such use is necessary, this VRA201 and the other equipment should be observed to verify that they are operating normally
- Use of accessories, transducers and cables other than those specified or provided by the manufacturer of this equipment could result in increased electromagnetic emissions or decreased electromagnetic immunity of this equipment and result in improper operation. The list of accessories, transducers and cables can be found in this appendix.
- Portable RF communications equipment (including peripherals such as antenna cables and external antennas) should be used no closer than 30 cm (12 inches) to any part of this VRA201, including cables specified by the manufacturer. Otherwise, degradation of the performance of this equipment could result in improper operation

To ensure compliance with the EMC requirements as specified in IEC 60601-1-2, it is essential to use only the following accessories:

Item	Manufacturer	Model
Patient response switch	Radioear	APS3

Conformance to the EMC requirements as specified in IEC 60601-1-2 is ensured if the cable types and cable lengths are as specified below:

Description	Length (m)	Screened (Yes/No)
Patient response switch	2.0	Yes

#### Guidance and manufacturer's declaration - electromagnetic emissions

The *VRA201* is intended for use in the electromagnetic environment specified below. The customer or the user of the *VRA201* should assure that it is used in such an environment.

Emissions Test	Compliance	Electromagnetic environment - guidance
RF emissions CISPR 11	Group 1	The <i>VRA201</i> uses RF energy only for its internal function. Therefore, its RF emissions are very low and are not likely to cause any interference in nearby electronic equipment.
RF emissions CISPR 11	Class B	The <i>VRA201</i> is suitable for use in all commercial, industrial, business, and residential environments.
Harmonic emissions IEC 61000-3-2	Complies Class A Category	
Voltage fluctuations / flicker emissions IEC 61000-3-3	Complies	

**Recommended separation distances between portable and mobile RF communications equipment and the VRA201.**

The **VRA201** is intended for use in an electromagnetic environment in which radiated RF disturbances are controlled. The customer or the user of the **VRA201** can help prevent electromagnetic interferences by maintaining a minimum distance between portable and mobile RF communications equipment (transmitters) and the **VRA201** as recommended below, according to the maximum output power of the communications equipment.

Rated Maximum output power of transmitter [W]	Separation distance according to frequency of transmitter [m]		
	150 kHz to 80 MHz $d = 1.17\sqrt{P}$	80 MHz to 800 MHz $d = 1.17\sqrt{P}$	800 MHz to 2.7 GHz $d = 2.23\sqrt{P}$
0.01	0.12	0.12	0.23
0.1	0.37	0.37	0.74
1	1.17	1.17	2.33
10	3.70	3.70	7.37
100	11.70	11.70	23.30

For transmitters rated at a maximum output power not listed above, the recommended separation distance  $d$  in meters (m) can be estimated using the equation applicable to the frequency of the transmitter, where  $P$  is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer.

**Note 1** At 80 MHz and 800 MHz, the higher frequency range applies.

**Note 2** These guidelines may not apply to all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.

**Guidance and Manufacturer's Declaration - Electromagnetic Immunity**

The **VRA201** is intended for use in the electromagnetic environment specified below. The customer or the user of the **VRA201** should assure that it is used in such an environment.

Immunity Test	IEC 60601 Test level	Compliance	Electromagnetic environment - guidance
Electrostatic Discharge (ESD) IEC 61000-4-2	± 8 kV contact ± 15 kV air	± 8 kV contact ± 15 kV air	Floors should be wood, concrete or ceramic tile. If floors are covered with synthetic material, the relative humidity should be greater than 30%.
Electrical fast transient/burst IEC61000-4-4	± 2 kV for power supply lines 100 kHz repetition frequency ± 1 kV Line-to-line 100 kHz repetition frequency	± 2 kV ± 1 kV	Mains power quality should be that of a typical commercial or residential environment.
Surge IEC 61000-4-5	± 1 kV Line-to-line ± 2 kV Line-to-ground	± 1 kV ± 2 kV	Mains power quality should be that of a typical commercial or residential environment.
Voltage dips, short interruptions and voltage variations on power supply lines IEC 61000-4-11	0% UT for 0.5 cycle 0 % UT for 1 cycle and 70% UT for 25/30 cycles Single phase: at 0°	0% UT for 0.5 cycle 0 % UT for 1 cycle and 70% UT for 25/30 cycles Single phase: at 0°	Mains power quality should be that of a typical commercial or residential environment. If the user of the <b>VRA201</b> requires continued operation during power mains interruptions, it is recommended that the <b>VRA201</b> be powered from an uninterruptable power supply or its battery.

Power frequency (50/60 Hz)  IEC 61000-4-8	30 A/m	30 A/m	Power frequency magnetic fields should be at levels characteristic of a typical location in a typical commercial or residential environment.
<b>Note:</b> <i>UT</i> is the A.C. mains voltage prior to application of the test level.			

**Guidance and manufacturer's declaration — electromagnetic immunity**

The **VRA201** is intended for use in the electromagnetic environment specified below. The customer or the user of the **VRA201** should assure that it is used in such an environment.

Immunity test	IEC / EN 60601 test level	Compliance level	Electromagnetic environment – guidance
Conducted RF IEC / EN 61000-4-6	3 Vrms 150kHz to 80 MHz	3 Vrms	Portable and mobile RF communications equipment should be used no closer to any parts of the <b>VRA201</b> , including cables, than the recommended separation distance calculated from the equation applicable to the frequency of the transmitter.  <b>Recommended separation distance:</b>  $d = 1,2\sqrt{P}$
	6 Vrms in ISM bands 150kHz to 80 MHz 80 % AM at 1 kHz	6 Vrms	
Radiated RF IEC / EN 61000-4-3	3 V/m 80 MHz to 2,7 GHz 80 % AM at 1 kHz	3 V/m	$d = 1,2\sqrt{P}$ 80 MHz to 800 MHz $d = 2,3\sqrt{P}$ 800 MHz to 2,7 GHz Where <i>P</i> is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer and <i>d</i> is the recommended separation distance in meters (m).  Field strengths from fixed RF transmitters, as determined by an electromagnetic site survey, <sup>a</sup> should be less than the compliance level in each frequency range. <sup>b</sup>  Interference may occur in the vicinity of equipment marked with the following symbol:  

**Note 1** At 80 MHz and 800 MHz, the higher frequency range applies

**Note 2** These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.

<sup>a)</sup> Field strengths from fixed transmitters, such as base stations for radio (cellular/cordless) telephones and land mobile radios, amateur radio, AM and FM radio broadcast and TV broadcast cannot be predicted theoretically with accuracy. To assess the electromagnetic environment due to fixed RF transmitters, an electromagnetic site survey should be considered. If the measured field strength in the location in which the **VRA201** is used exceeds the applicable RF compliance level above, the **VRA201** should be observed to verify normal operation, If abnormal performance is observed, additional measures may be necessary, such as reorienting or relocating the **VRA201**.

<sup>b)</sup> Over the frequency range of 150 kHz to 80 MHz, field strengths should be less than 3 V/m.

## NOTICE

ESSENTIAL PERFORMANCE for this VRA201 is defined by the manufacturer as:

- This VRA201 does not have an ESSENTIAL PERFORMANCE Absence or loss of ESSENTIAL PERFORMANCE cannot lead to any unacceptable immediate risk
- Final diagnosis shall always be based on clinical knowledge
- There are no deviations from the collateral standard and allowances uses
- This VRA201 complies with IEC60601-1-2:2014, emission class B group 1

## NOTICE

All necessary instructions for maintaining compliance regarding EMC can be found in the general maintenance section in this Instructions for Use. No further steps required.

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