


Instructions for Use  
**OtoAccess<sup>®</sup> ADI**

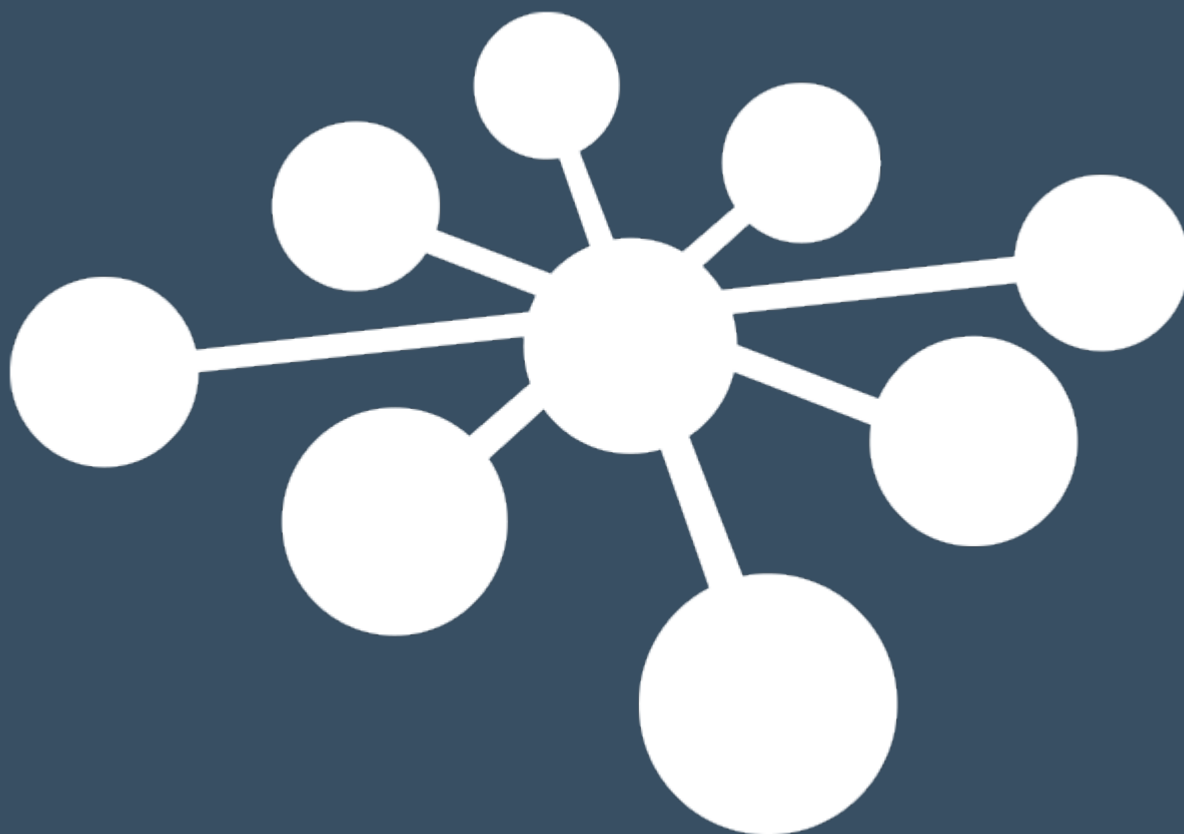

# Table of Contents

<b>1</b>	<b>Introduction .....</b>	<b>1</b>
1.1	About this Manual .....	1
1.2	The ADI overview.....	1
1.3	Implementation .....	2
<b>2</b>	<b>License .....</b>	<b>3</b>
<b>3</b>	<b>Installation and configuration.....</b>	<b>4</b>
3.1	PC requirements .....	4
3.2	API .....	4
3.3	Legacy support.....	4
<b>4</b>	<b>Documentation.....</b>	<b>5</b>
<b>5</b>	<b>Registering hook .....</b>	<b>6</b>
5.1	Description .....	6
5.2	Sequence diagram .....	6
<b>6</b>	<b>Getting list of registered applications .....</b>	<b>7</b>
6.1	Description .....	7
6.2	Sequence diagram .....	7
<b>7</b>	<b>Setting current patient .....</b>	<b>8</b>
7.1	Description .....	8
7.2	Sequence diagram .....	8
<b>8</b>	<b>Launching registered application .....</b>	<b>9</b>
8.1	Description .....	9
8.2	Sequence diagram .....	9
<b>9</b>	<b>Retrieving data from application .....</b>	<b>10</b>
9.1	Description .....	10
9.2	Sequence diagram .....	11
<b>10</b>	<b>Retrieving printouts.....</b>	<b>12</b>
10.1	Description .....	12
10.2	Sequence diagram .....	12
<b>11</b>	<b>Sending saved session back to application for review.....</b>	<b>13</b>
11.1	Description .....	13
11.2	Sequence diagram .....	14
<b>12</b>	<b>Working with standalone instruments .....</b>	<b>15</b>
12.1	Connect/Disconnect instruments .....	15
12.1.1	Description.....	15
12.1.2	Sequence diagram .....	15
12.2	Add/Remove patients from instruments.....	16
12.2.1	Description.....	16
12.2.2	Sequence diagram .....	16
12.3	Get sessions.....	17
12.3.1	Description.....	17
12.3.2	Sequence diagram .....	17
12.4	Get tests data.....	18
12.4.1	Description.....	18

12.4.2	Sequence diagram .....	18
<b>13</b>	<b>Setting clinic and examiner .....</b>	<b>19</b>
13.1	Description .....	19
13.2	Sequence diagram .....	19


# 1 Introduction

## 1.1 About this Manual

This manual is valid for the Audiometric Data Interface. The product is manufactured by:  
OtoAccess A/S  
Audiometer Allé 1  
5500 Middelfart  
Denmark

## 1.2 The ADI overview

The Audiometric Data Interface (ADI) is an interface between the audiological hardware and an EMR system. The ADI provides a pathway to enable staff at hospital and healthcare practices to securely store and retrieve data from the EMR system. Manufacturers can use it to seamlessly integrate supported standalone audiological hardware to the many EMR systems, enabling an efficient, streamlined user workflow.

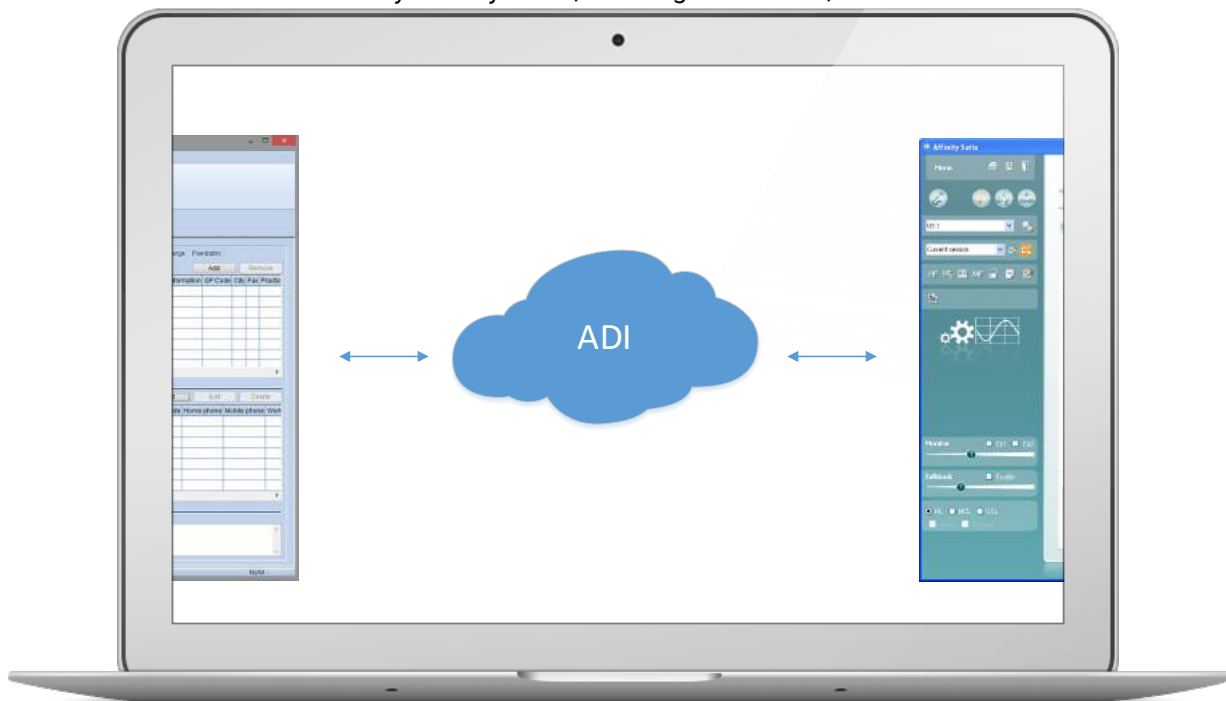

**Figure 1: The ADI is placed between EMR system and audiological software/hardware.**



### 1.3 Implementation

The current ADI is implemented as a local (not networked) self-hosted WebAPI and it is installed as a windows service. It expose communication via REST api which is well-documented and easily accessed for the 3rd party manufacturers.

This document describes how to use ADI to communicate with supported products via an HTTP-based RESTful API. The provided API will work with software modules or direct connection to a standalone instrument.

Usages:

- Send/Retrieve Patient demographics
- Send/Retrieve Sessions
- Delete patient/sessions

**Note:**

This interface is not intended for controlling the software or the instruments, but to exchange patient information and sessions.



## 2 License

### **Important!**

PC Sync or PC control license is in some cases needed in order to be able to connect to the instruments via ADI. Instruments which are not license controlled will connect with ADI.



## 3 Installation and configuration

### 3.1 PC requirements

- Windows® 7 SP1, Windows® 8.x or Windows® 10
- .NET 4.6.2

### 3.2 API

The RESTful API is installed as a Windows® Service.

Installation process requires using Command Line or Power Shell, where `msiexec` command is executed. Server and printer port numbers are configurable while installation process with `PORT_NUMBER_PRINTER` and `PORT_NUMBER_SERVER` arguments. Port number must be number from 1 to 65535 included. Logging while installing is highly recommended. Example of command executed in folder, where installation file occurs:

```
msiexec /i "OtoAccess ADI Server Setup.msi" PORT_NUMBER_PRINTER="X" ^ PORT_NUMBER_SERVER="Y" /!* "log.txt"
```

Port numbers are not required, default values are: 9100 for printer and 47294 for server. Please refer to the **readme.txt** file for a full list of configurable parameters.

### 3.3 Legacy support

Interfacing to legacy modules (e.g. Eclipse and VNG) is achieved by installing the `laBaseProxy` application.

Legacy applications lack the ability to save reports automatically to the API, and instead requires the user to print reports manually to the ADI Printer that is installed as part of the API.

As API, `laBaseProxy` must be installed via `msiexec` command. List of installed modules is configurable while installation process with `MODULES` argument. Available modules: `Teoae`, `Vng`, `Dpoae`, `Ep`, `Assr` and `Abris`. Logging while installing is highly recommended. Example of command executed in folder, where installation file occurs:

```
msiexec /i "ADI laBaseProxy Setup.msi" MODULES="Assr,Vng" ^  
INSTALLFOLDER="C:\ExampleFolder\ExampleSubFolder" /!* "log.txt"
```

`INSTALLFOLDER` is not required, default: `C:\Program Files (x86)\OtoAccess\Audiometric Data Interface`. `MODULES` argument cannot be empty.

Please refer to the **readme.txt** file for a full list of configurable parameters.



## 4 Documentation

Once installed, the API provides an interactive help page at the following address: [http://localhost:PORT\\_NUMBER/help](http://localhost:PORT_NUMBER/help). It is based on reliable Swagger RESTful API documentation specification. The documentation site allows client to check available API operations with short description and multiple response content type. There is a possibility to test methods via “TRY IT OUT!” option. To get access to Server’s data, provided api\_key is required.

PORT\_NUMBER is configured during installation (see section [API](#)) and default is 47294.

Note that the port number can be specified during installation and might differ from the above address.

In order to open the OtoAccess® ADI demo database with the documentation site please press the apikey ‘ApiKey Public’.





## 5 Registering hook

### 5.1 Description

A hook is a mechanism by which an application can intercept events, such as messages like POST and GET HTTP methods. It allows for two-way communication between Client and ADI.

A hook can be registered via query the ADI via POST [/hooks?portNumber=XYZ](#), where XYZ is a port number.

Swagger doc: [http://URL/help/index#!/Hooks/Hooks\\_AddHook](http://URL/help/index#!/Hooks/Hooks_AddHook).

### 5.2 Sequence diagram

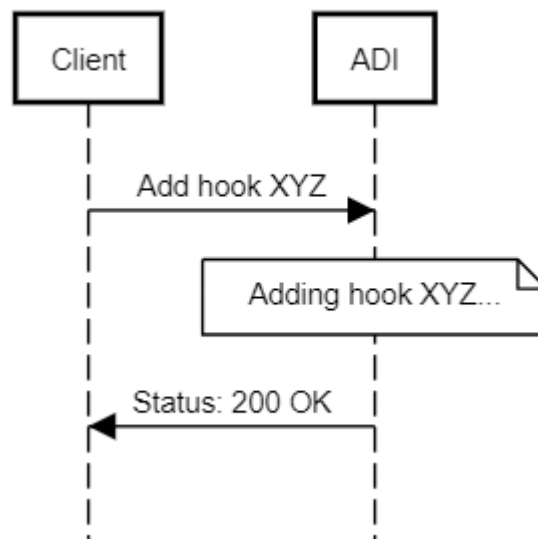


Figure 2 Adding hook



## 6 Getting list of registered applications


### 6.1 Description

The registered applications are suites and modules installed on the given machine. In order to list them, query the ADI via GET [/applications](#). In response list of application will be attached.

Note that every application contains detailed information like: Filename, Arguments etc.

Swagger doc: [http://URL/help/index#!/Applications/Applications\\_GetApplications](http://URL/help/index#!/Applications/Applications_GetApplications)

### 6.2 Sequence diagram

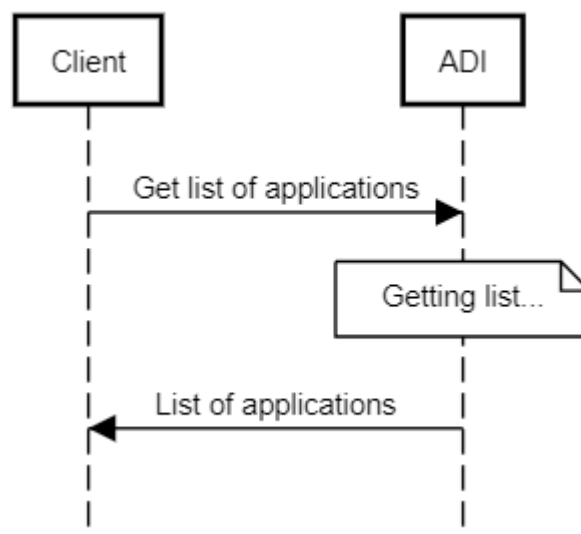

**Figure 3** Getting list of registered applications




## 7 Setting current patient

### 7.1 Description

To work with suites or modules, current patient have to be set. To do that send patient to ADI via POST /patient.

Swagger doc: [http://URL/help/index#!/Patient/Patient\\_Post](http://URL/help/index#!/Patient/Patient_Post)

### 7.2 Sequence diagram

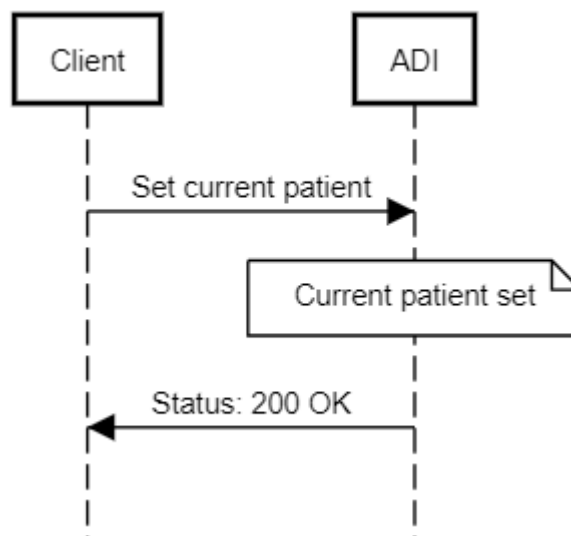

*Figure 4 Setting current patient*



## 8 Launching registered application

### 8.1 Description

To launch application, the current patient has to be set already (see section 7). Next step is to find and execute registered application (see section 6). As a result, suite or module should launch with patient data.

Swagger doc:

[http://URL/help/index#!/Applications/Applications\\_GetApplications](http://URL/help/index#!/Applications/Applications_GetApplications)

[http://URL/help/index#!/Patient/Patient\\_Post](http://URL/help/index#!/Patient/Patient_Post)

### 8.2 Sequence diagram

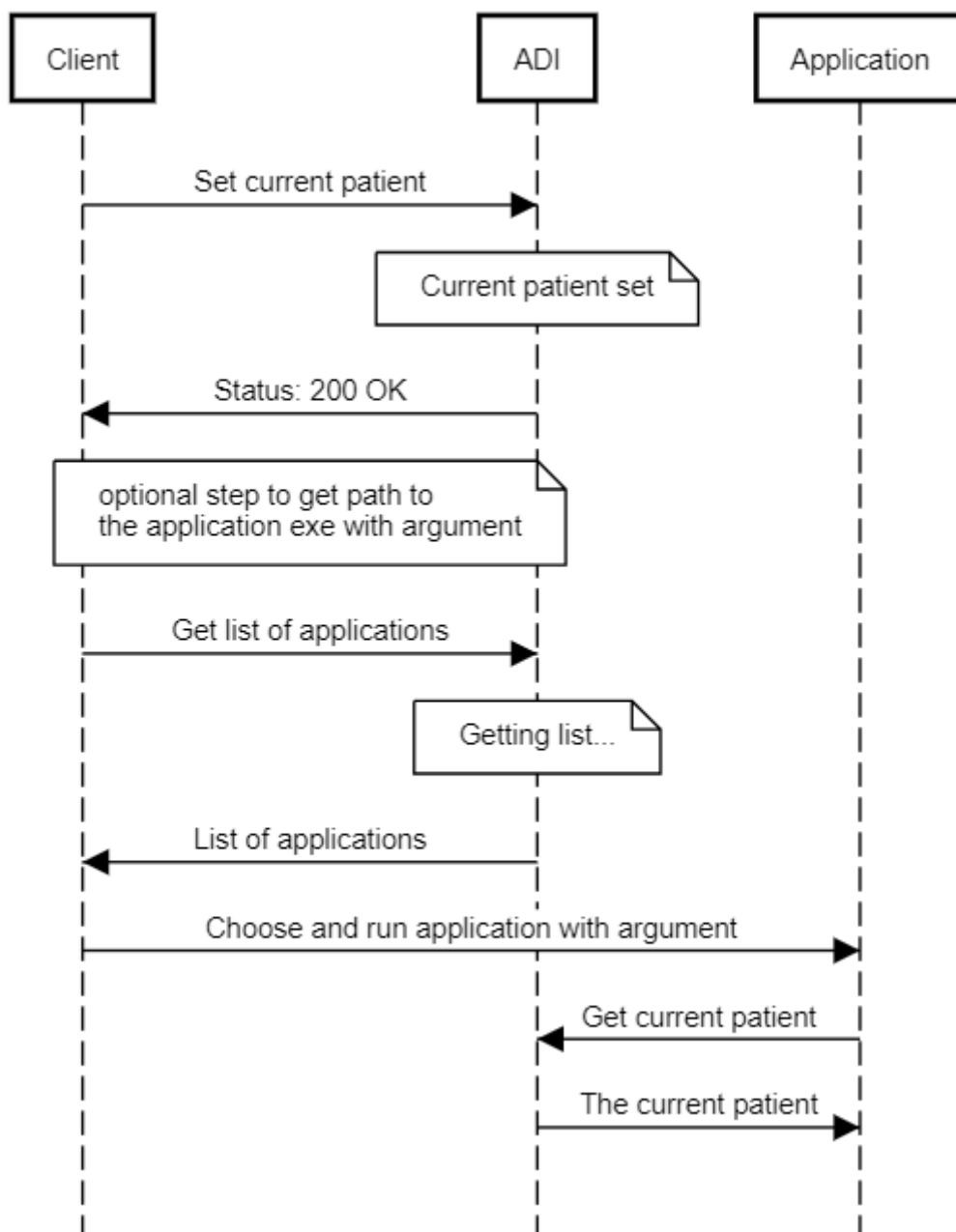


Figure 5 Launching registered application



## 9 Retrieving data from application

### 9.1 Description

When data is available, suite or module will send notification under registered hook (see section 5). The client is responsible for retrieving or cleaning data. To get list of available data, query ADI via GET /sessions, in response you will get list of saved sessions. Then data is available by reference to a specific session's index.

**Please note that the report data are only available for the PC Suites (see relevant brand specific technical note). In order to take it from Legacy Modules please see section 10 and relevant brand specific technical note).**

In order to retrieve:

- Binary data  
GET /sessions/{index}
- Test data  
GET /sessions/{index}/tests
- Report data  
GET /sessions/{index}/report
- Image data  
GET /sessions/{index}/image

In order to clean:

- DELETE /sessions/{index}

Swagger doc:

[http://URL/help/index#!/Sessions/Sessions\\_Get](http://URL/help/index#!/Sessions/Sessions_Get)

[http://URL/help/index#!/Sessions/Sessions\\_Get\\_0](http://URL/help/index#!/Sessions/Sessions_Get_0)

[http://URL/help/index#!/Sessions/Sessions\\_GetTests](http://URL/help/index#!/Sessions/Sessions_GetTests)


[http://URL/help/index#!/Sessions/Sessions\\_GetReport](http://URL/help/index#!/Sessions/Sessions_GetReport)

[http://URL/help/index#!/Sessions/Sessions\\_GetImage](http://URL/help/index#!/Sessions/Sessions_GetImage)

[http://URL/help/index#!/Sessions/Sessions\\_Delete](http://URL/help/index#!/Sessions/Sessions_Delete)



## 9.2 Sequence diagram

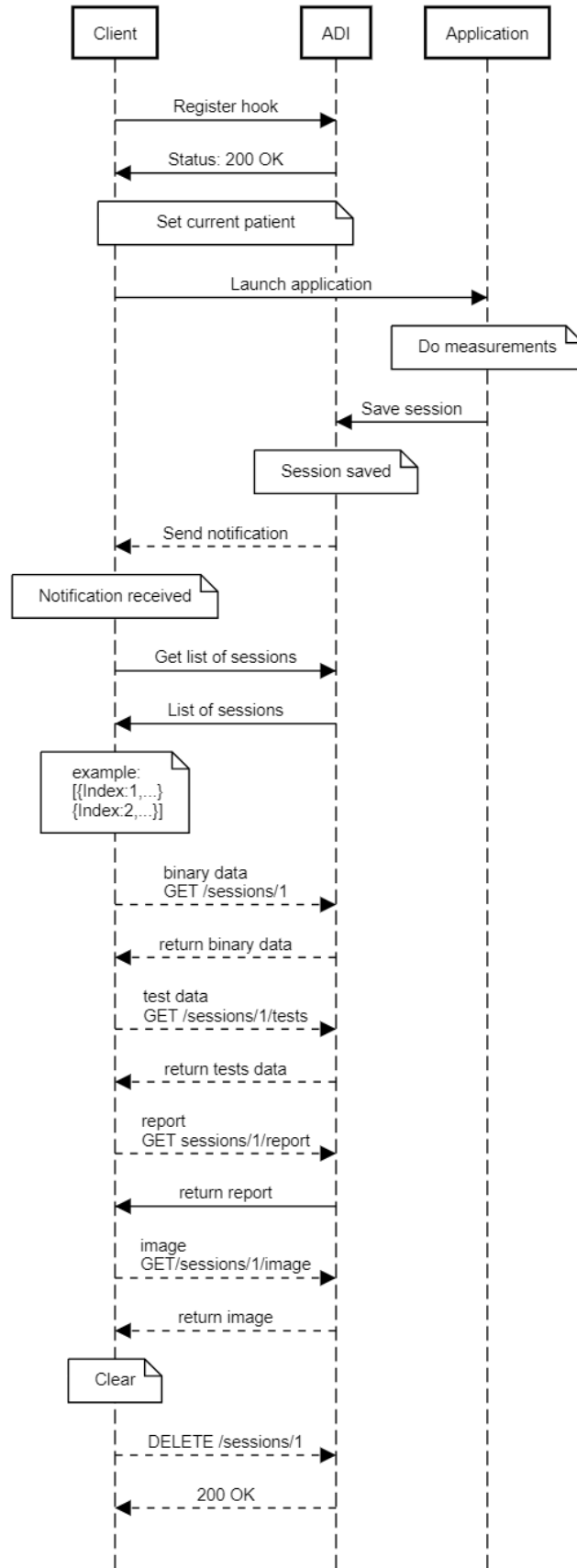

**Figure 6** Retrieving data from application



# 10 Retrieving printouts

## 10.1 Description

In order to list printouts, query the ADI via GET [/printout](#). To get specified report, the used query is GET [/printout/indexnumber/report](#) with specified report's index number.

**Please note that 'printouts' are the reports from the Legacy Modules which use printer to send report data. See relevant brand specific technical note.**

Swagger doc:  
<http://localhost:47294/help/index#!/Printout>

## 10.2 Sequence diagram

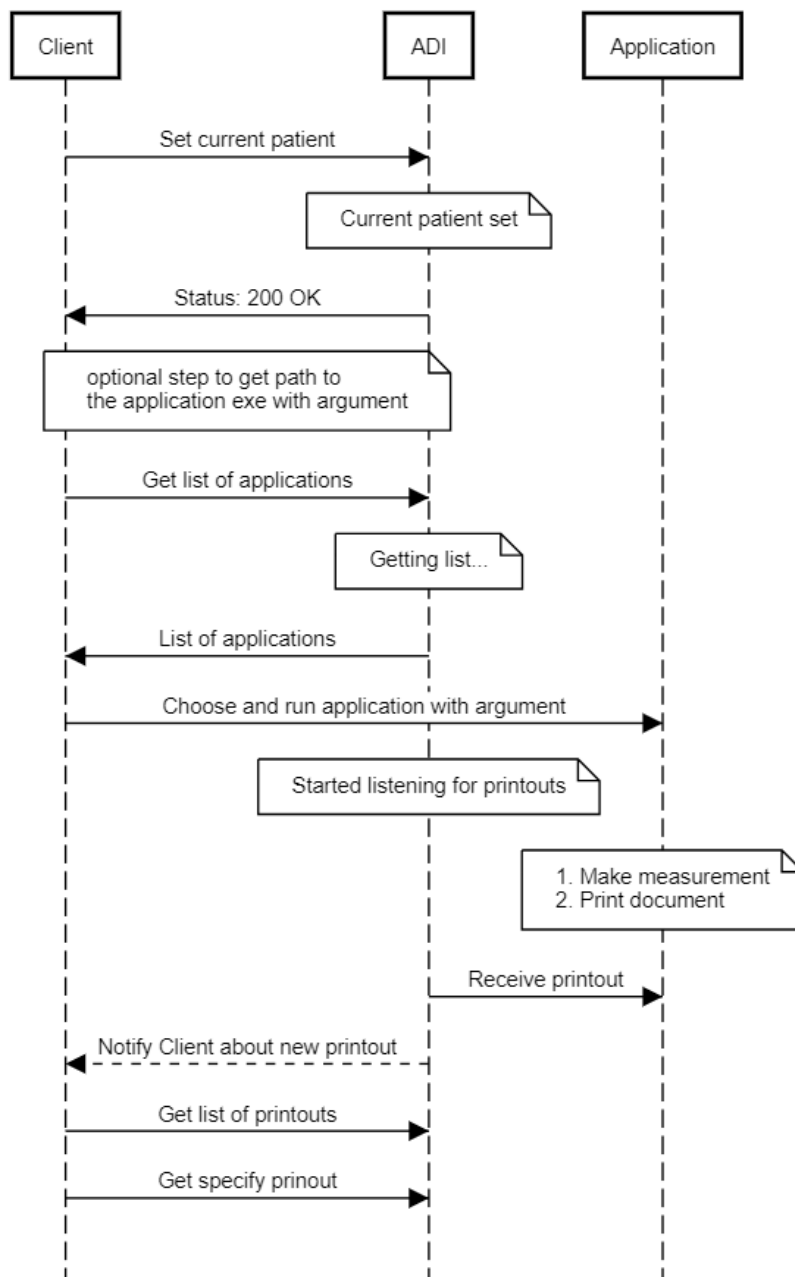


Figure 7 Retrieving printouts



## 11 Sending saved session back to application for review

### 11.1 Description

To be able to review sessions within application, Client has to set the current patient with attached sessions. In order to do that Client retrieve session and match it with proper patient by 'GUID' property (Patient) and 'PatientGuid' (Session). Next attach proper session binary data to the patient's 'Sessions' property and set patient as current patient into ADI.

Swagger doc:

[http://URL/help/index#!/Patient/Patient\\_Get](http://URL/help/index#!/Patient/Patient_Get)

[http://URL/help/index#!/Patient/Patient\\_Post](http://URL/help/index#!/Patient/Patient_Post)

[http://URL/help/index#!/Sessions/Sessions\\_Get](http://URL/help/index#!/Sessions/Sessions_Get)

[http://URL/help/index#!/Sessions/Sessions\\_Get\\_0](http://URL/help/index#!/Sessions/Sessions_Get_0)





### 11.2 Sequence diagram

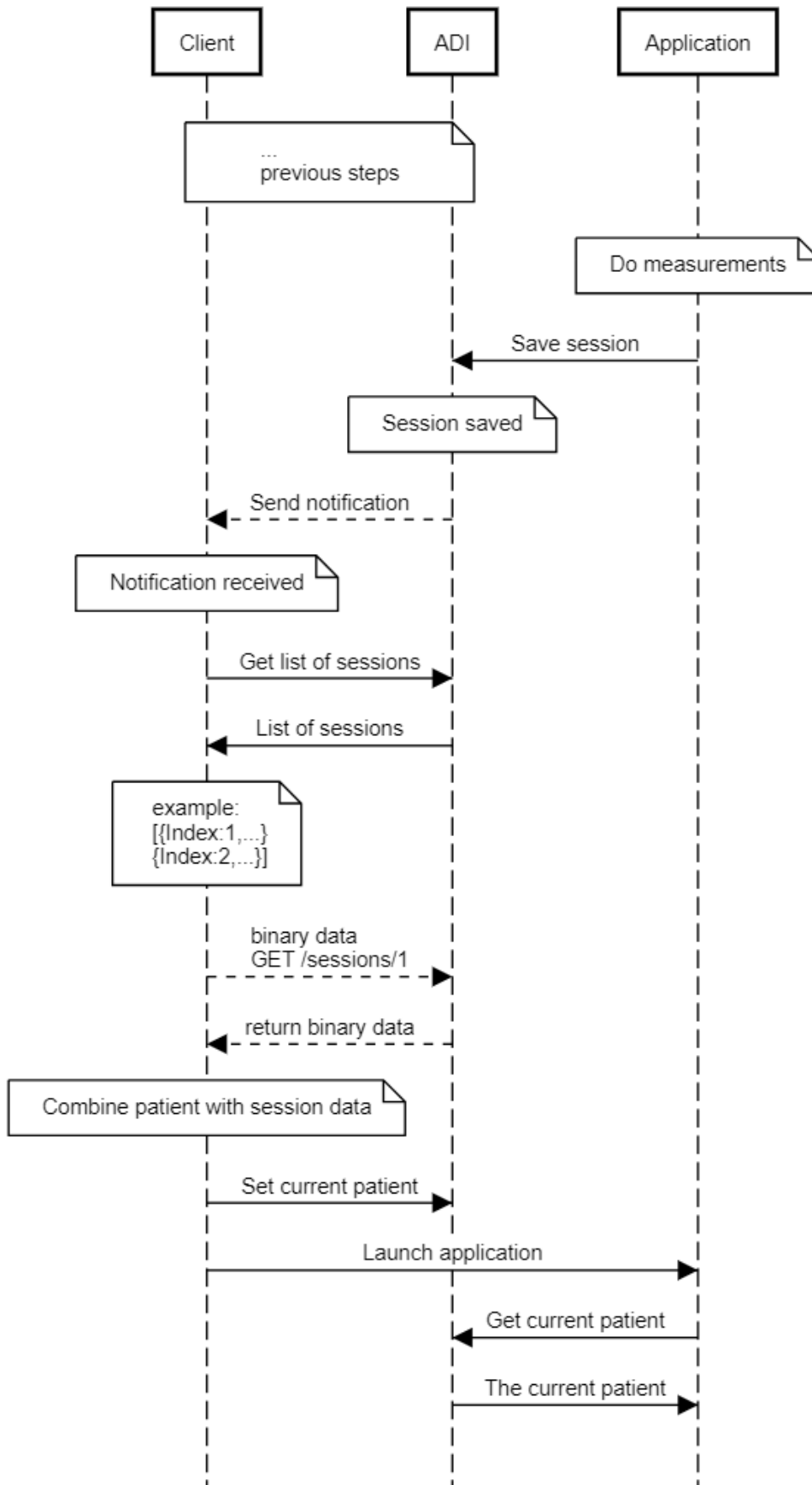


Figure 8 Sending saved session back to application



## 12 Working with standalone instruments

### 12.1 Connect/Disconnect instruments

#### 12.1.1 Description

The ADI is also working well with standalone instruments. Supported instruments are listed in section 4.1. After plug in via cable, it should be immediately visible on the list of instruments GET /instruments. In order to access patients and sessions, it is required to connect to the instrument. You can send POST request under /instruments/{index}/connect (to disconnect use /instruments/{index}/disconnect).

Swagger doc:

[http://URL/help/index#!/Instruments/Instruments\\_GetInstruments](http://URL/help/index#!/Instruments/Instruments_GetInstruments)

[http://URL/help/index#!/Instruments/Instruments\\_Connect](http://URL/help/index#!/Instruments/Instruments_Connect)

[http://URL/help/index#!/Instruments/Instruments\\_Disconnect](http://URL/help/index#!/Instruments/Instruments_Disconnect)

#### 12.1.2 Sequence diagram

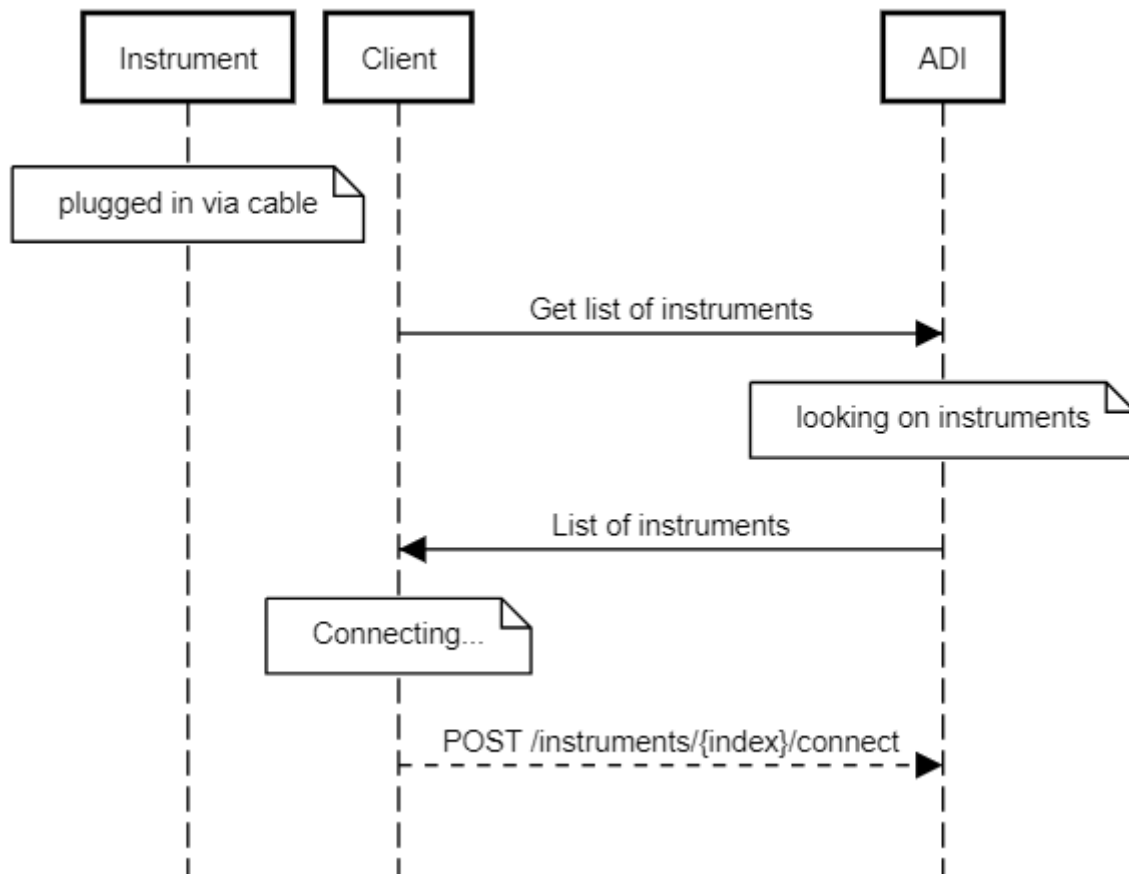


Figure 9 Connecting to the instrument



## 12.2 Add/Remove patients from instruments

### 12.2.1 Description

After connecting the instrument (see section 14.1), you can check which patients are currently on the instrument by query ADI via GET /instruments/{index}/patients. It is also possible to set own patient via POST /instruments/{index}/patients (to remove patient send DELETE /instruments/{instrumentIndex}/patients/{patientIndex}).

Swagger doc:

[http://URL/help/index#!/Instruments/Instruments\\_GetPatients](http://URL/help/index#!/Instruments/Instruments_GetPatients)

[http://URL/help/index#!/Instruments/Instruments\\_AddPatient](http://URL/help/index#!/Instruments/Instruments_AddPatient)

[http://URL/help/index#!/Instruments/Instruments\\_DeletePatient](http://URL/help/index#!/Instruments/Instruments_DeletePatient)

### 12.2.2 Sequence diagram

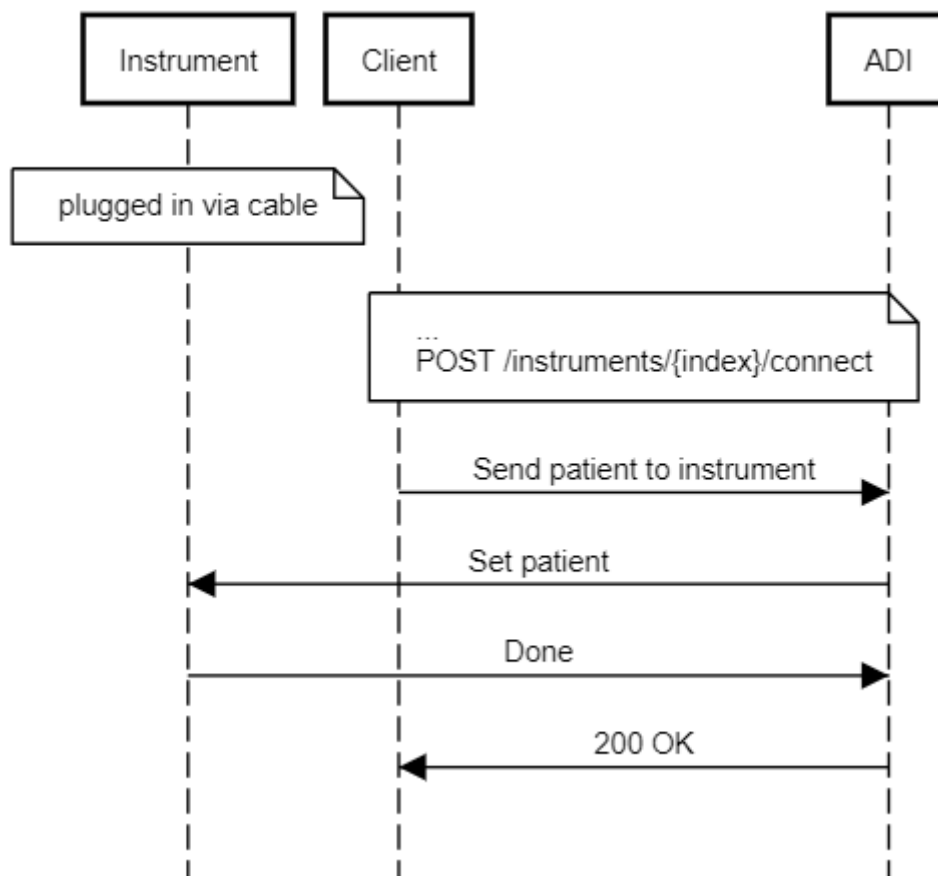


Figure 10 Add patient to the instrument



### 12.3 Get sessions

#### 12.3.1 Description

The sessions are available by query ADI via GET /instruments/{instrumentIndex}/{patientIndex}/sessions. Please note that the instrument has to be connected and patient is accessible at given index (see section 14.1 and 14.2).

Swagger doc:

[http://URL/help/index#!/Instruments/Instruments\\_GetSessions](http://URL/help/index#!/Instruments/Instruments_GetSessions)

#### 12.3.2 Sequence diagram

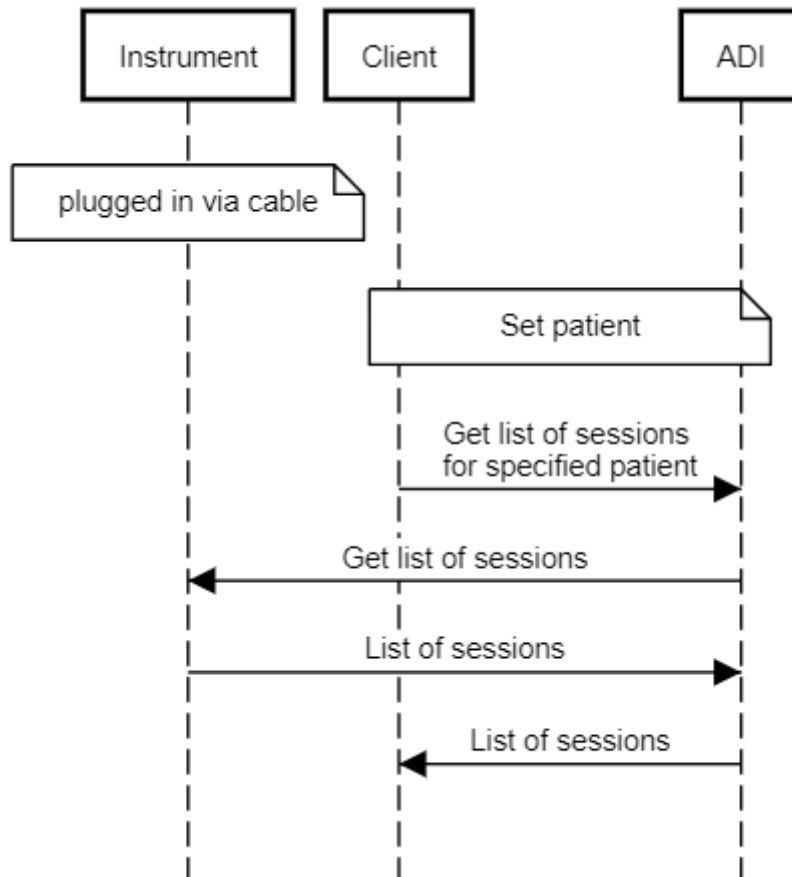


Figure 11 Get list of sessions for specified patient



## 12.4 Get tests data

### 12.4.1 Description

The tests data which is stored in the session can be retrived by query ADI via /instruments/{instrumentIndex}/patients/{patientIndex}/sessions/{sessionIndex}/tests. The requirements are that the instrument is connected, and patient with sessions is set.

Swagger doc:  
[http://URL/help/index#!/Instruments/Instruments\\_GetTests](http://URL/help/index#!/Instruments/Instruments_GetTests)

### 12.4.2 Sequence diagram

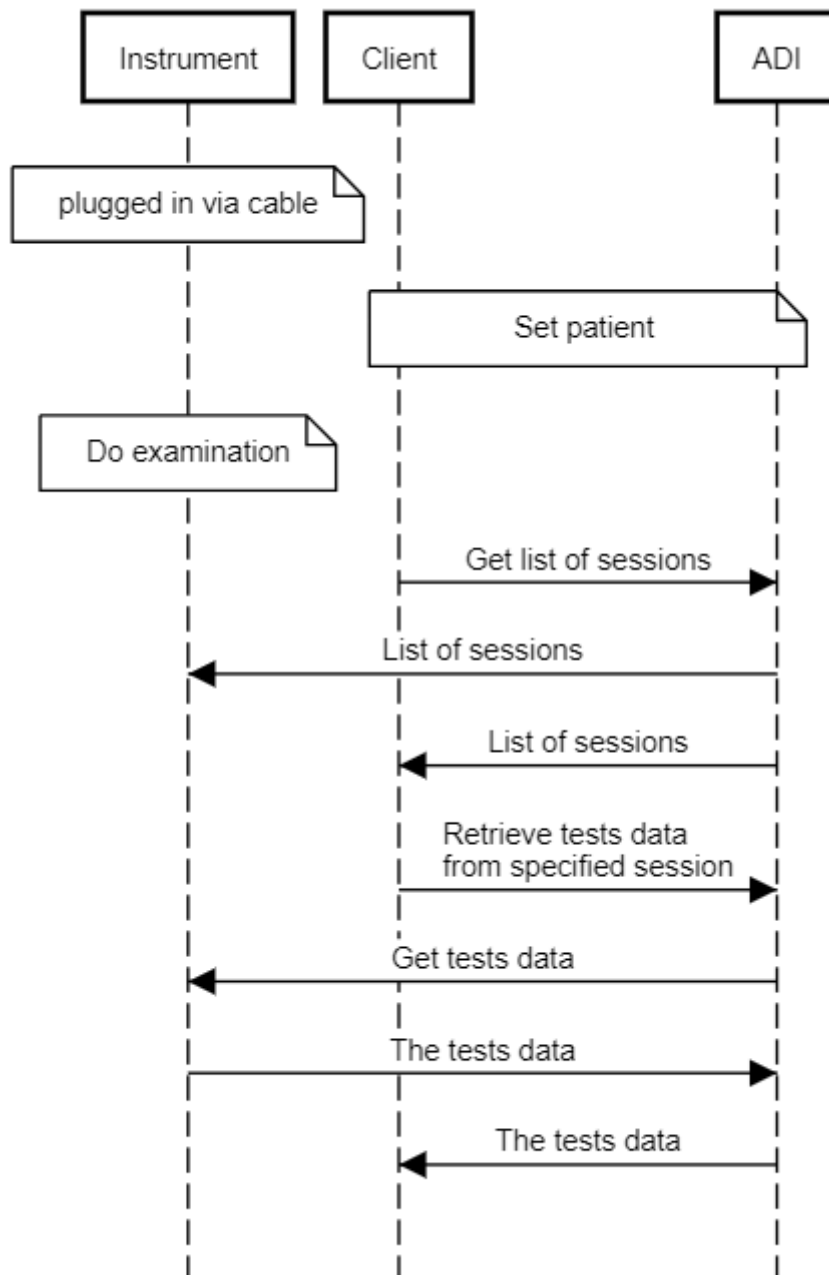


Figure 12 Get tests data



## 13 Setting clinic and examiner

### 13.1 Description

The client is responsible for setting clinic and examiner information. In order to do that, please POST data under /clinic and /examiner.

Swagger doc:

[http://URL/help/index#!/Clinic/Clinic\\_Get](http://URL/help/index#!/Clinic/Clinic_Get)

[http://URL/help/index#!/Clinic/Clinic\\_Post](http://URL/help/index#!/Clinic/Clinic_Post)


[http://URL/help/index#!/Clinic/Clinic\\_Delete](http://URL/help/index#!/Clinic/Clinic_Delete)

[http://URL/help/index#!/Examiner/Examiner\\_Get](http://URL/help/index#!/Examiner/Examiner_Get)

[http://URL/help/index#!/Examiner/Examiner\\_Post](http://URL/help/index#!/Examiner/Examiner_Post)

[http://URL/help/index#!/Examiner/Examiner\\_Delete](http://URL/help/index#!/Examiner/Examiner_Delete)

### 13.2 Sequence diagram

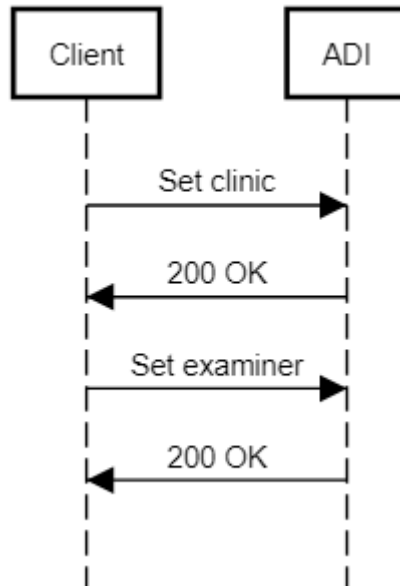

*Figure 13 Setting clinic and examiner*

