



Waves (Sway Referenced)

Module tutorial



Table of contents

1.	GENERAL	3
1.1.	Description	3
1.2.	Required accessories.....	3
1.3.	Patient setup	3
2.	Settings	3
2.1.	Session settings	3
3.	Module.....	4
3.1.	Session	4
3.2.	Shortcuts	5
3.3.	Results	5



1. GENERAL

1.1. Description

WAVES (SWAY REFERENCED) is an immersive 3D simulation software based on virtual reality technology, meaning a person can be immersed in a digitally created artificial world. The software is used for desensitization in cases of naupathia (sea sickness).

1.2. Required accessories

VR headset.


1.3. Patient setup

It depends on the therapist's goals.

The patient can be seated in a chair or seated on a swivel chair, with feet off the floor, so that the practitioner can perform angular accelerations (rocking movements and random half-turns are sufficient).


2. Settings

2.1. Session settings

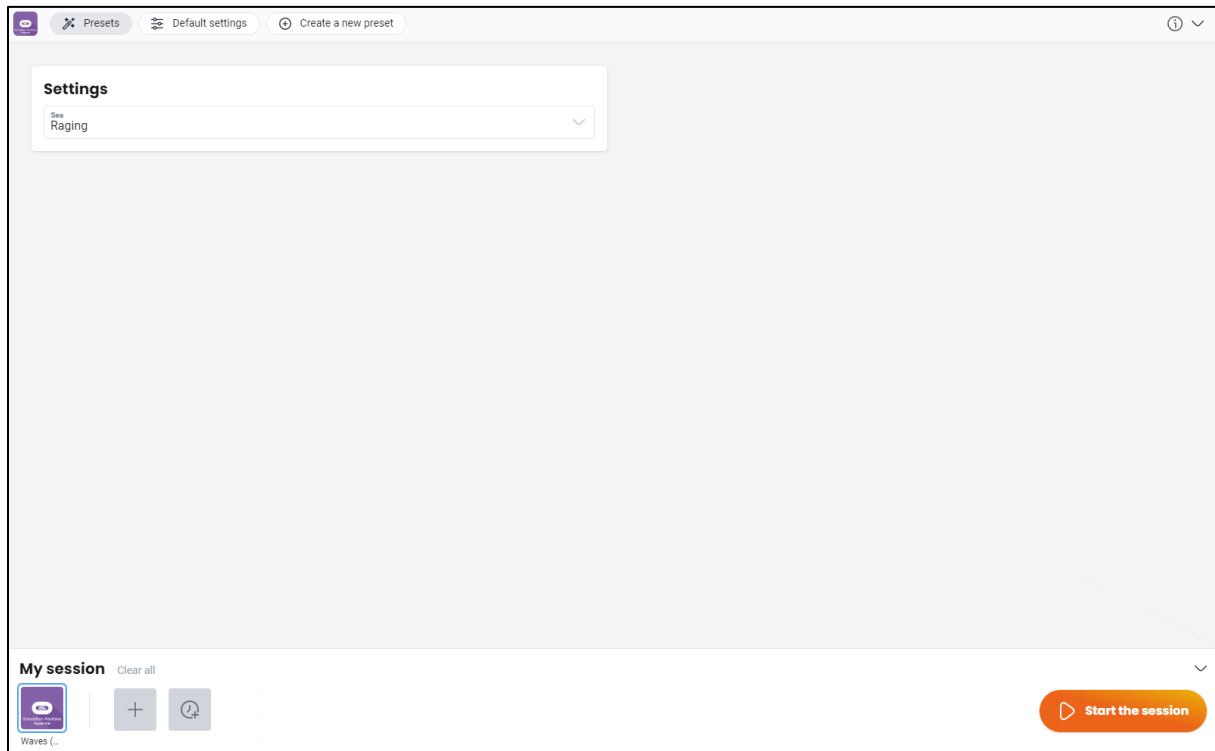

Settings:


The type of sea can be selected (raging, stormy, calm, flat) depending on patient sensitivity.

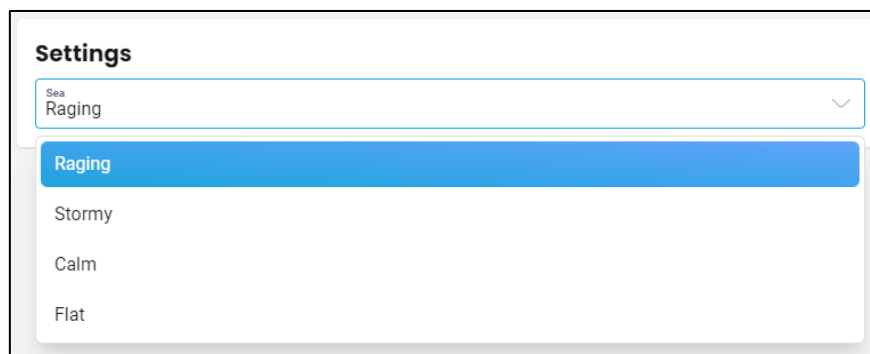

3. Module

3.1. Session


Once the presets have been selected, click on "**Start the session**" in the bottom right corner of the screen.

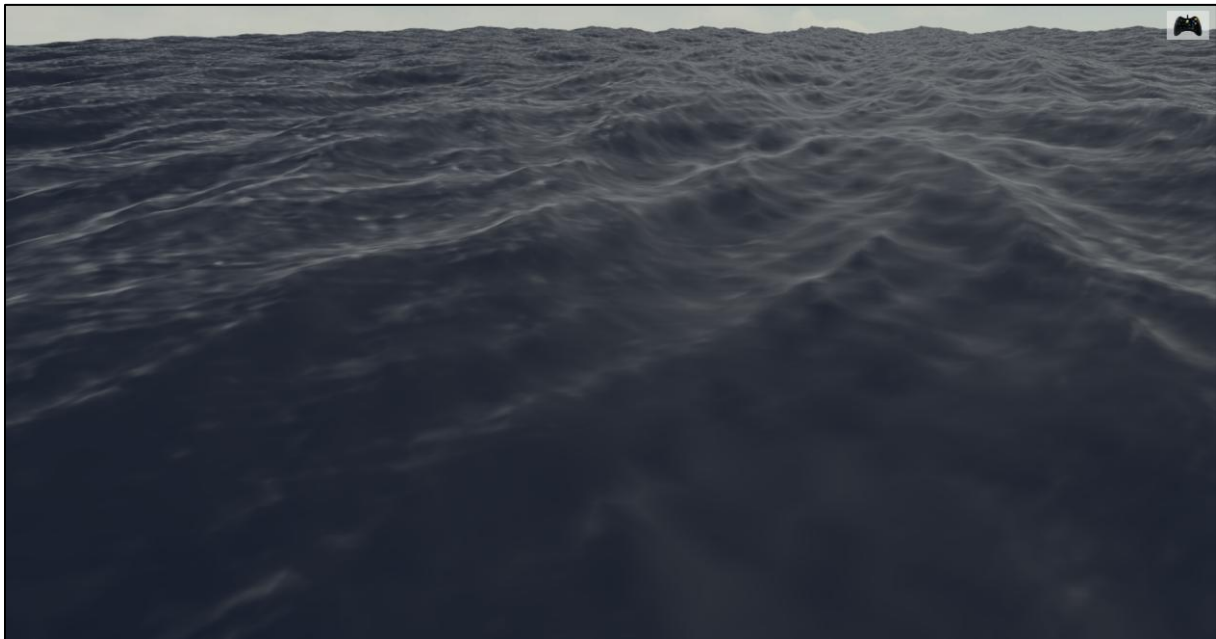

End the session by pressing the “**Escape**” key on the keyboard.

3.2. Shortcuts

During the session, the shortcut list is found by clicking on the Xbox controller icon in the upper right corner of the screen.

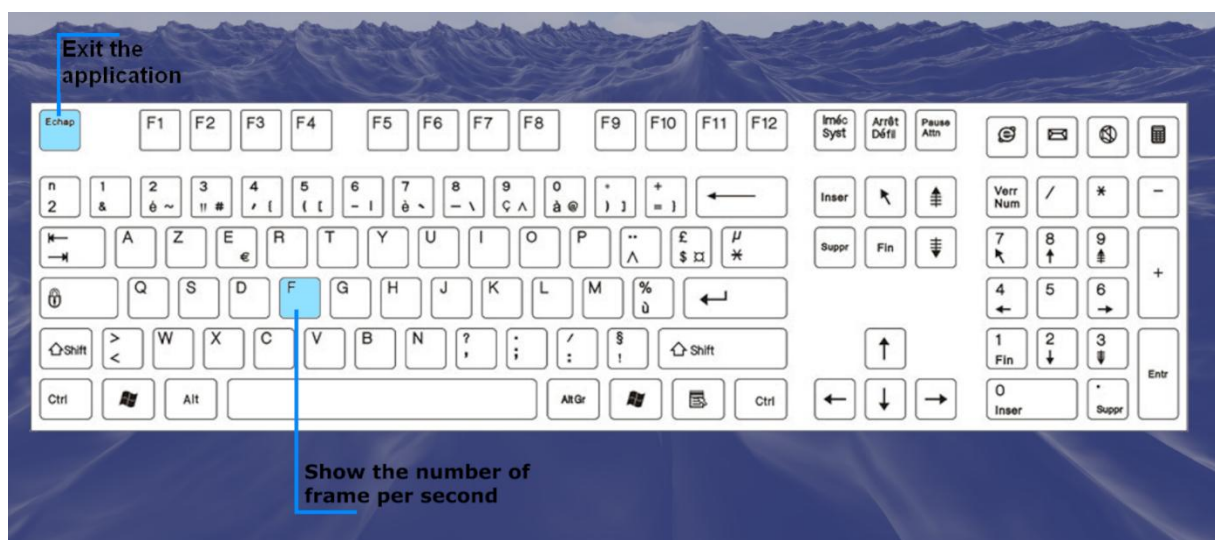

3.3. Results



No results are displayed at the end of the session.